



The Careva Kombi KIT can be fitted in all seats/chairs that have a solid backrest and an opening between the seat and backrest. All Careva products are positioning belts. An upright sitting position is required in order for the regular safety belt in vehicles to work optimally.



Safety first! All Careva products are positioning belts. Always use the regular safety seat belt together with a positioning belt in vehicles.

CE Approval LVFS 2003 : 11, LVFS 2001 : 7, LVFS 2001 : 5

Careva COMBI KIT User manual article no. 30223 (S), 30224 (M), 30225 (L)

Open the plastic packaging and take out the plastic plate that the belt is fixed to. Loosen the velcro fastenings and release the belt from the plate. Inspect the product and read the user instructions.

1. Open the two plastic clips that hold the spinal belt together and fix the spinal belt to the back support of the chair. Lock the belt to the back of the back support.
2. Adjust the spinal belt vertically so that the shoulder section of the combi harness is adjusted to the shoulder height of the person using the harness.
3. Make sure that the lap belt is positioned in the channel at the bottom of the seat.
4. Open the plastic clips on the shoulder harness and the lap belt and unfold the lap belt on the seat.
5. Place the person on top of the crotch strap on the chair and open the padded section over the pelvis. Lock on each side. Pull so that the harness secures the pelvis.
6. Place the shoulder straps over the shoulders, clip the shoulder straps on the front of the lap belt. Adjust by pulling on the straps.

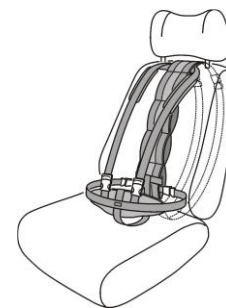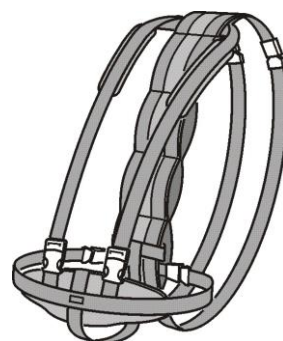

Careva AB
Therese Svenssons Gata 10
SE 417 55 Gothenburg
Sweden
Phone +46 31 222800
www.careva.se
info@careva.se

