



The Careva Belt system can be fitted in all seats/chairs that have a solid backrest and an opening between the seat and backrest. All Careva products are positioning belts. An upright sitting position is required in order for the regular safety belt in vehicles to work optimally.

Safety first! All Careva products are positioning belts. Always use the regular safety seat belt together with a positioning belt in vehicles.

CE Approval LVFS 2003 : 11, LVFS 2001 : 7, LVFS 2001 : 5

Careva Shoulder-KIT user manual, article no. 30121 (S), 30020 (M), 30033 (L)

The shoulder Kit is used to prevent the person from falling forwards or sideways.

1. Open the plastic packaging and take out the plastic plate that the belt is fixed to. Loosen the velcro fastenings and release the belt from the plate. Inspect the product and read the user instructions.
2. Place the spinal belt around the back support and lock it at the back. Adjust so that the channels are at the front of the back support and tighten the strap(s) so that the harness is securely fastened.
3. Adjust to the person's shoulder height. Ensure that the velcro fastenings are attached to each other inside the channel (to prevent the harness from sliding sideways).
4. A G-lock, item no 10030(300 mm long S) or 10040(400 mm long M/L) is pre-mounted in the channel immediately below. Ensure that the velcro fastenings are attached to each other inside the channel (to prevent the G-lock from sliding sideways).
5. Place the padded sections over the person's shoulders. Lock the shoulder harness on both sides of the G-lock under the arms. Tighten. To avoid the shoulder harness sliding down, you can, where appropriate, attach an extra chest lock across the chest (item no. 10122(S) or 10123 (M/L).

Careva AB
Theres Svenssons gata 10
S 41755 Gothenburg
Sweden
+46 (0) 31 222800
info@careva.se
www.careva.se



